

SonicCloud: Mac Installation Guide


Installing SonicCloud

First, download the installer file for SonicCloud personalization from:

<https://www.soniccloud.com/download>


You'll need to scroll down to the bottom of the page to download the SonicCloud for Mac link.

Once you have the file, installation is straightforward. Just double click the .pkg file you downloaded, and follow the instructions.

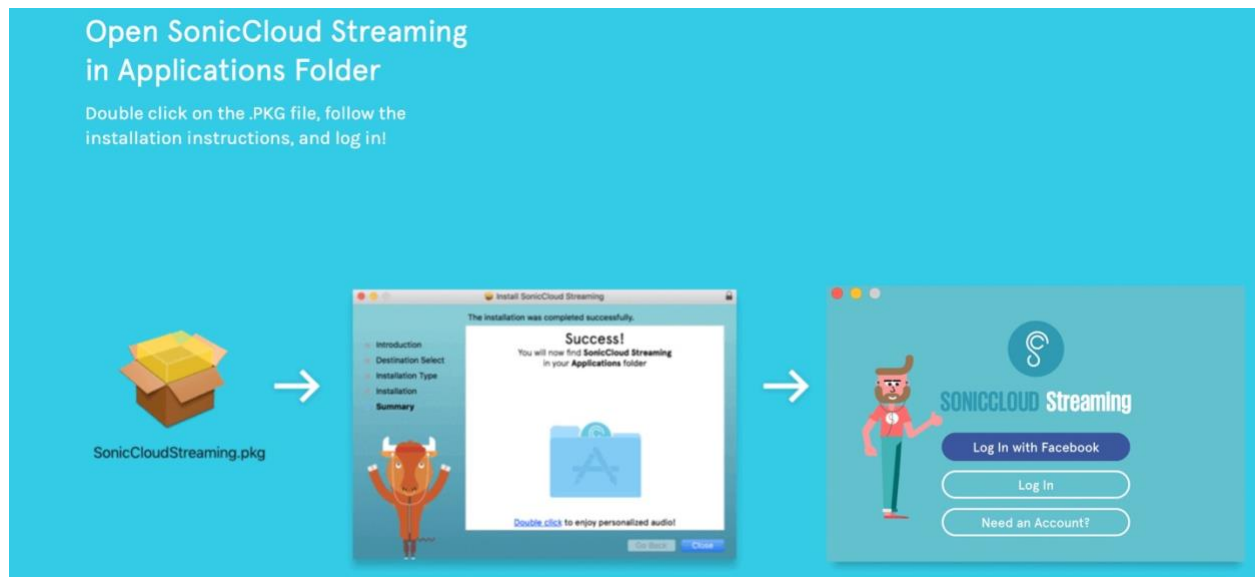


Setup


After installation, confirm that SonicCloud has been selected as the audio device on your machine. Select “System Preferences” on your dock.

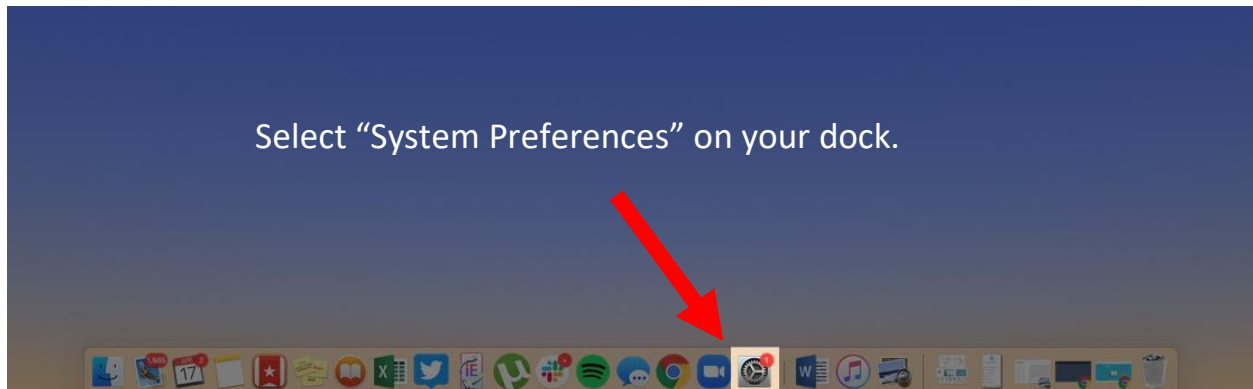


If you don't see “System Preferences” on your dock, you can search for it under the magnifying glass on the top right-hand corner of your screen.

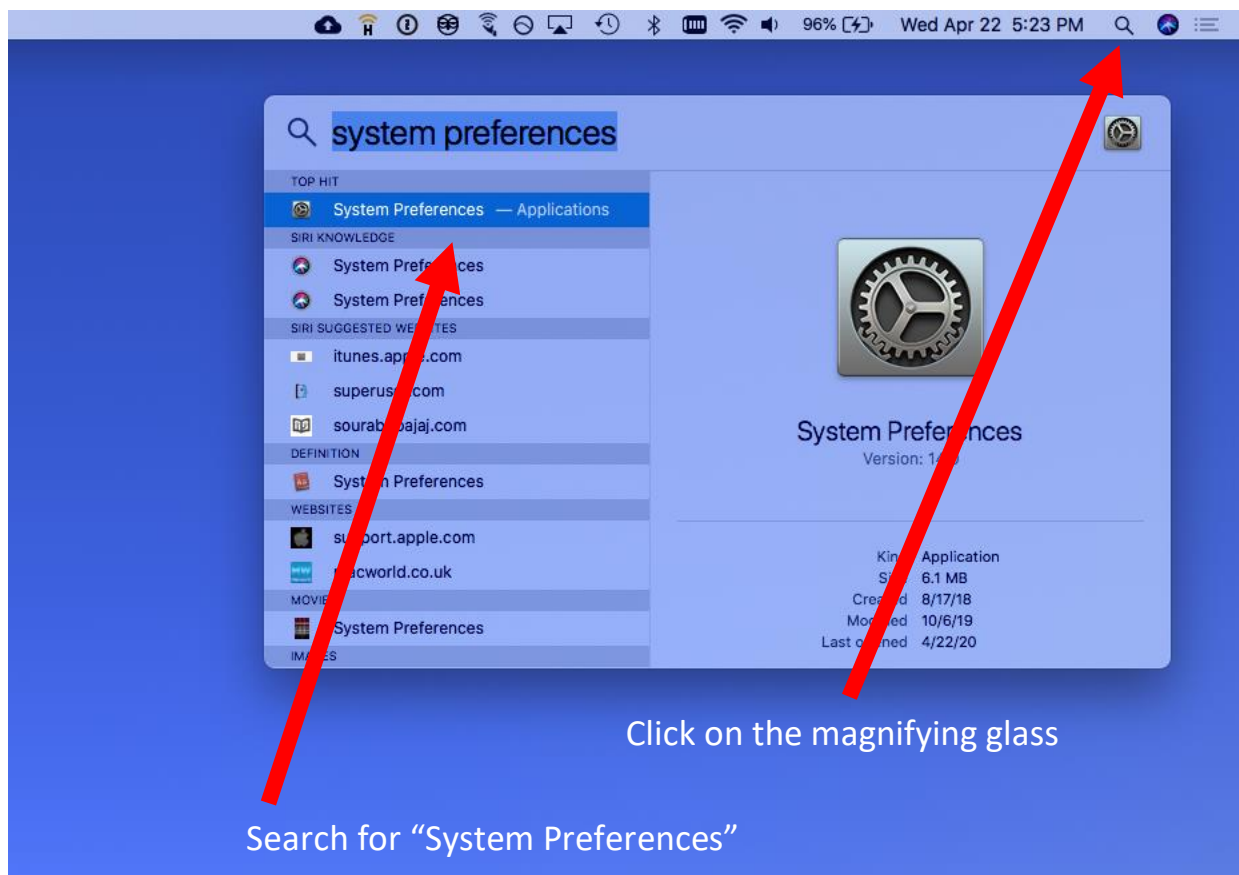

Select "Sound" under System Preferences.

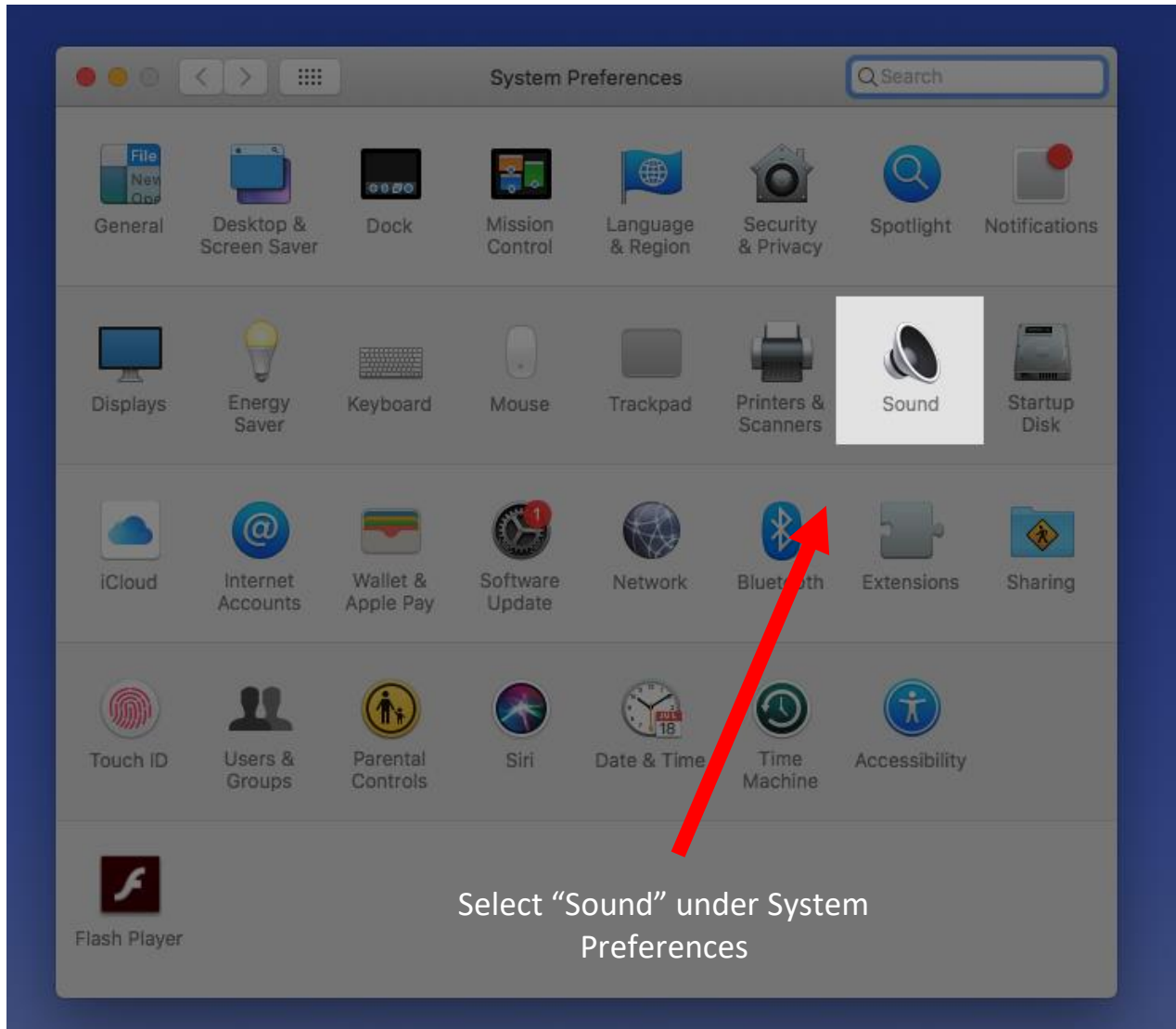


Select the checkbox for “Show volume in menu bar”

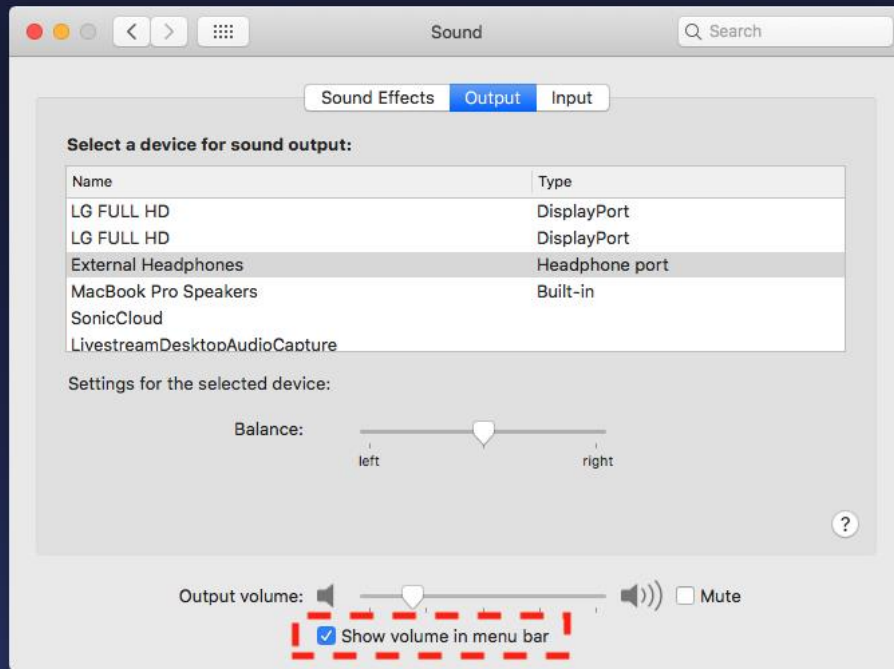


Select the checkbox for “Show volume in menu bar”

Confirm “SonicCloud is selected under the volume icon on your top menu bar!

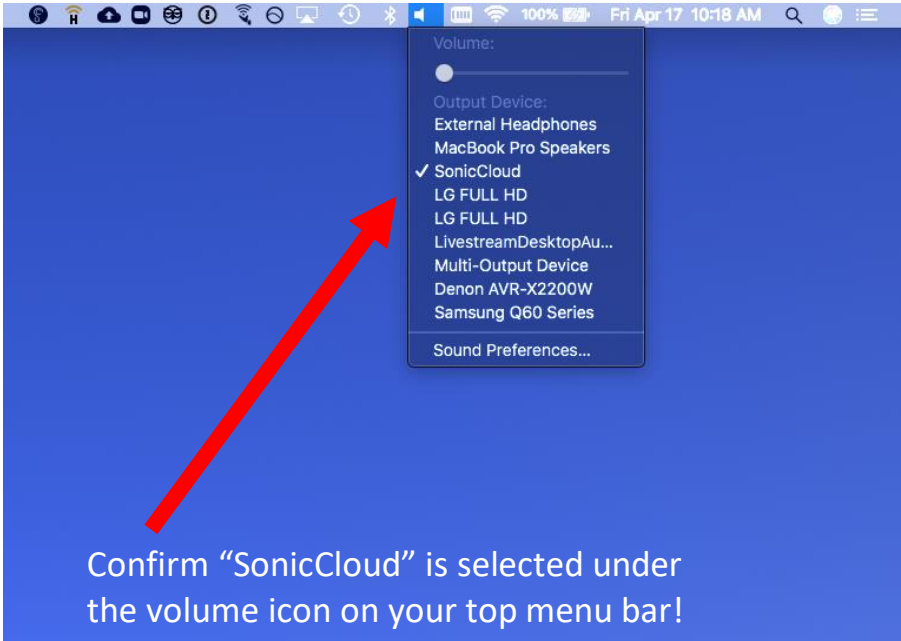


Log In to SonicCloud

After installing SonicCloud, the SonicCloud console will pop up in the bottom center of your screen. Then click on the **Log In** button in the middle of the SonicCloud console.

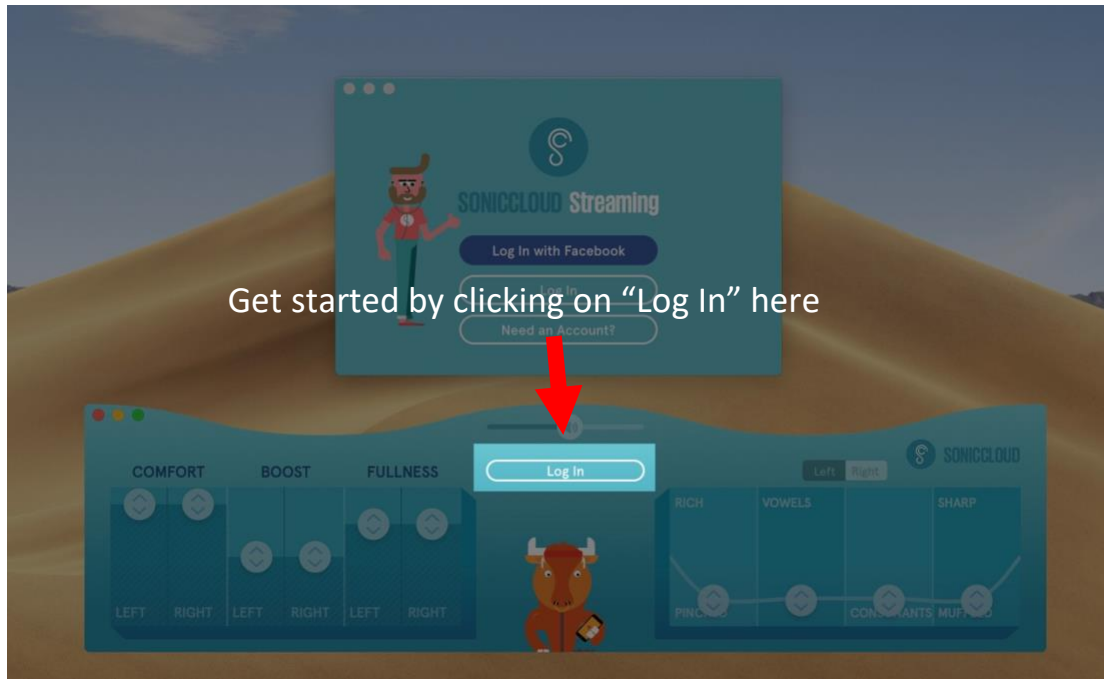

You will now see a window that lets you log in to your account.

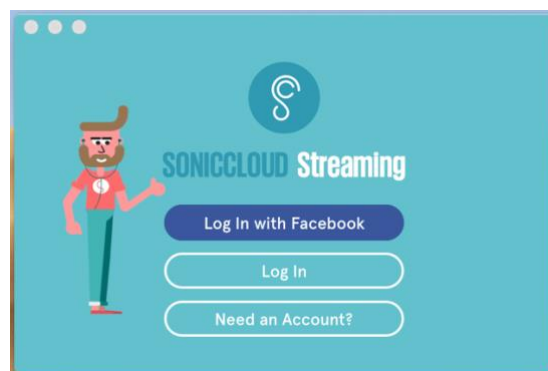


If you don't have a SonicCloud account

1. Download the SonicCloud app on your smartphone from [Google Play Store](#) (Android Phone) or [App Store](#) (iPhone)
2. Create a new account
3. Complete the Hearing Assessment
4. Once you've created an account on your smartphone and finished the hearing assessment, you can log in on your MacOS machine.

If you already have a SonicCloud account

If you already have a SonicCloud account, you can log in by selecting “Log In”, and entering your email and password.

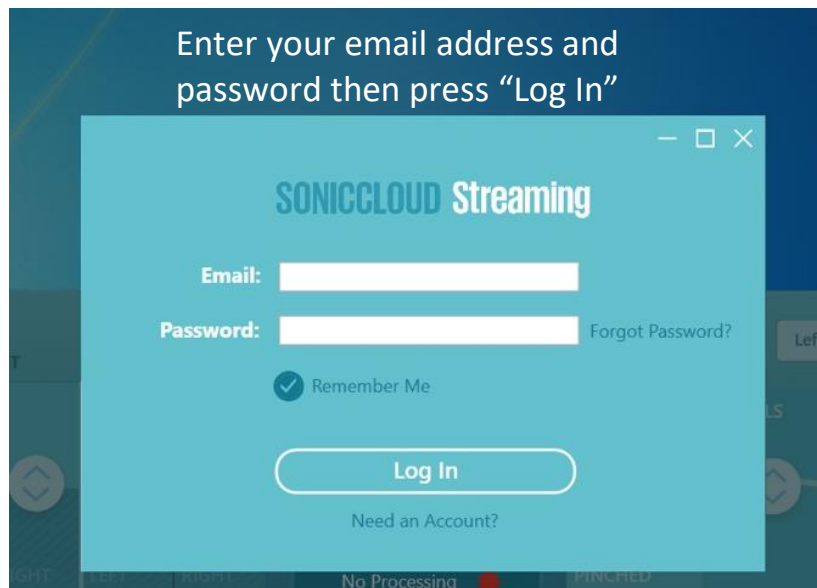

You've now completed all the configuration steps for SonicCloud on your computer!

