

1

Open outer peel pouch and transfer InQu PASTE MIX PLUS to sterile field within sealed tray.

1



2

In the mixing tray, combine InQu PASTE MIX PLUS powder with autograft*, and then gradually add blood, saline, or other fluid (1–2 cc at a time).

2



3


Thoroughly mix the PASTE MIX PLUS after each addition of fluid until desired consistency is achieved.

3



4


Examples of composite InQu graft: molded, on spatula, in a cage.

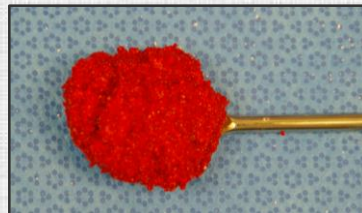

*Autograft sources could include: local bone, iliac crest, bone shavings, or milled bone

Preparation Overview

See reverse side for step-by-step mixing guide.

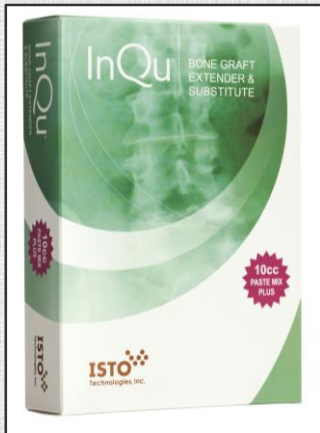

1. Ask the surgeon about the intended use of InQu PASTE MIX PLUS (PMP) before mixing, as desired consistency varies by application.
 - For putty consistency, add less blood or fluid
 - If flowable graft is desired, add more blood or fluid
2. If autograft (local bone, milled bone, or bone shavings) will be used, mix with InQu PMP powder **prior** to adding blood or fluid. Be careful not to overhydrate.
3. If bone shavings are collected from a trap, remove excess moisture by squeezing in a sterile cloth **prior** to mixing with InQu PMP powder and adding blood or fluid.
4. **Gradually** add blood or fluid – not more than 1-2 cc at a time – depending on volume of InQu used and desired consistency.