

INSTRUCTIONS FOR USE

CP-95 Face Mask (*Continuous Protection-95 Face Mask*)


For Personal Protective Equipment

This “Instruction for use” contains information on how to wear the CP-95 Face Mask properly.

Important information you need to know before wearing the CP-95 Face Mask

Step 1:


Wash your hands thoroughly before putting on and taking off the CP-95 Face Mask (**Figure A**).

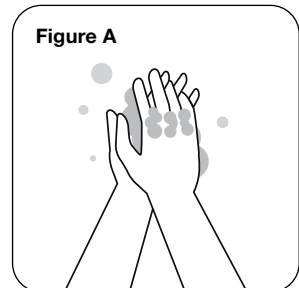

Step 2:


Inspect the face mask and filter packaging for damage. If your face mask or filters appear damaged, DO NOT USE IT. Replace it with a new one. Do not allow facial hair, hair, jewelry, glasses, clothing, or anything else to prevent proper placement or come between your face and the mask (**Figure B**).

Fix the elastic straps to the face mask shell. **See strap assembly instructions.**

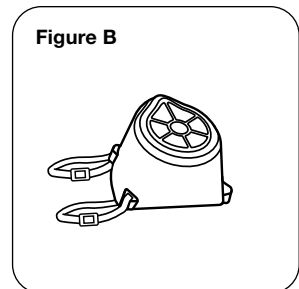

Step 3:

Sterilize the CP-95 Face Mask plastic materials, as needed, with sanitation baths, wipes, soaps, or steam.

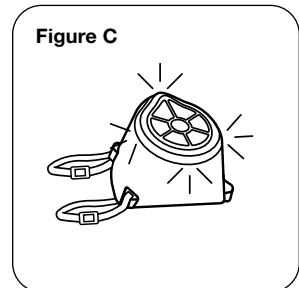


Step 4:

Open the filter cap on the front of the face mask and insert a new, unused filter. Close the filter cap until it is secured (**Figure C**). Ensure a proper seal of the filter cap.

Step 5:


Place the face mask over your nose and mouth. Be sure the nose seal is on top (**Figure D**).

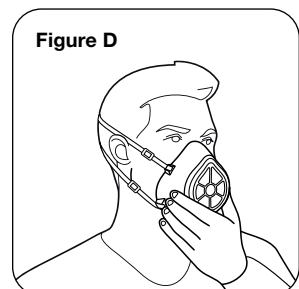

Instructions for use continued


Step 6:

Pull the top elastic strap over your head until it rests on the crown of your head above your ears (**Figure E**).

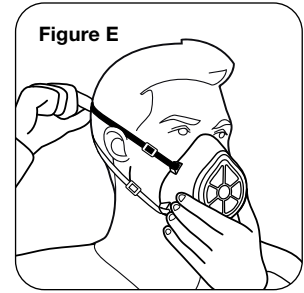


Step 7:

Pull the bottom elastic strap over your head until it rests just below your ears (**Figure F**).

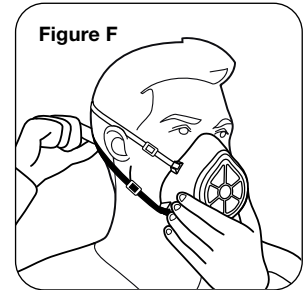


Step 8:

Using the adjustable clips on both elastic straps, tighten or loosen the straps to achieve a secure fit around your face and nose (**Figure G**).

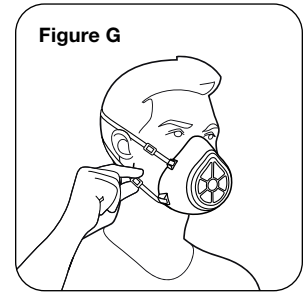

Step 9:

Using both hands starting at the top, press down on the edges of the face mask to achieve a secure seal. Use two hands (**Figure H**).

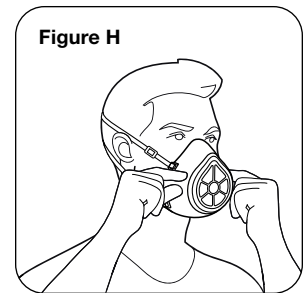

April 29, 2020

Declaration of Personal Use:

THE CP-95 FACE MASK HAS NOT BEEN FDA CLEARED OR APPROVED. THE CP-95 FACE MASK HAS BEEN AUTHORIZED BY THE FDA UNDER AN EMERGENCY USE AUTHORIZATION ("EUA") FOR USE AS A SOURCE CONTROL BY THE GENERAL PUBLIC AS WELL AS BY HEALTH CARE PERSONNEL IN HEALTH CARE SETTINGS TO HELP PREVENT THE SPREAD OF INFECTION OR ILLNESS ONLY FOR THE DURATION OF THE COVID-19 EUA.

Visit <https://www.cf3d.co/cp95-disclaimer> to view the full disclaimer.