

# 06 How do I submit a blood glucose reading?

Ref: CORP-059-01 Version 1 7-Dec-20

**Task:** Submit a reading using an AgaMatrix WaveSense Jazz Wireless meter

**Previous actions:** Pair an AgaMatrix WaveSense Jazz Wireless meter

## Step 1


Take a blood glucose reading as shown by your clinician.

## Step 2

Open the app, tap on the + button on the hub page.


Ensure that your meter is set up with the app.

Tap on **Take BG reading**.

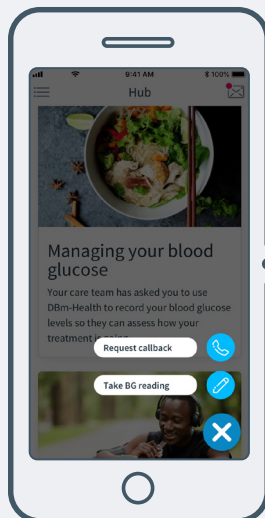


## Step 3

Ensure your meter is turned on then select **Scan** from the buttons presented.

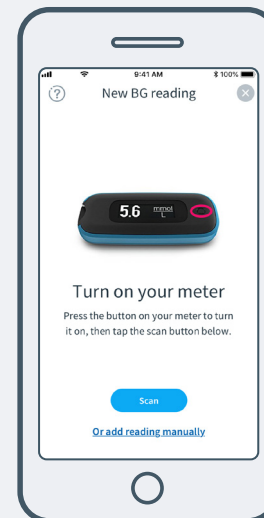

## Step 4

Your newest reading will come through on the enter BG reading page:

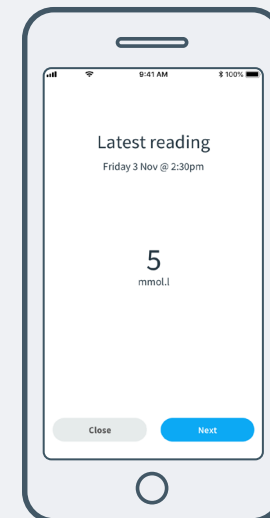


## Step 5

Select the meal type associated with the reading. If a reading is before a meal, you may be asked about the medication you have taken.

Type in the dose of the medication and tap **Next**.


**NOTE:** If you are not on medication, please proceed to **Step 7**.

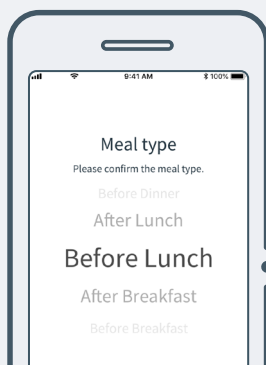

## Step 6


If you have not inputted a dose for a medication you will get an **Input required** warning.

Please review your dose and type in a number. If you have not taken your medication then please enter **0**.

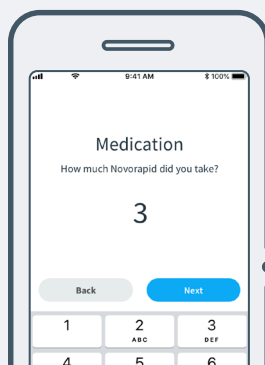

## Step 7

If you want to tell your diabetes care team something about your reading then please type this information into the **New Comment** screen and then submit.

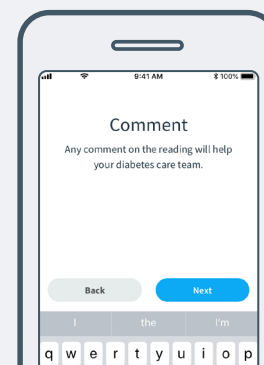

## Step 8

Your reading will then appear on your diary and graph screens

**NOTE:** If you started to enter a new reading unintentionally, tap the **Cancel** button.