

# 07 How do I submit a blood glucose reading?

Ref: CORP-059-01 Version 1 7-Dec-20

## Task: Submit a blood glucose reading manually

Previous actions taken: Set up own meter


### Step 1

Take a blood glucose reading as shown by your clinician with your meter.

### Step 2

Open the app, tap on the + button on the hub page.

Tap on **Take BG reading**.

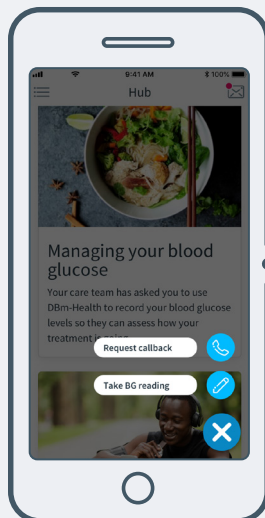

### Step 3


Press the input reading button.

### Step 4

Enter the blood glucose value which is displayed on your blood glucose meter.

Then tap **Next**.

**NOTE:** If you started to enter a new reading unintentionally, tap the **Close** button.

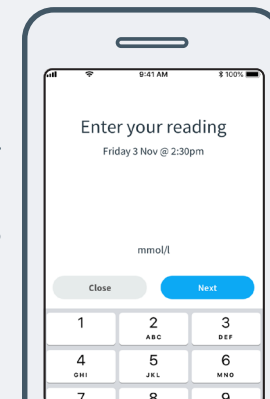


### Step 5

Select the meal type associated with the reading. If a reading is before a meal, you may be asked about the medication you have taken.

Type in the dose of the medication and tap **Next**.


**NOTE:** If you are not on medication, please proceed to **Step 7**.

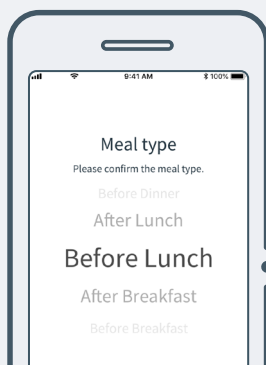

### Step 6


If you have not inputted a dose for a medication you will get an **Input required** warning.

Please review your dose and type in a number. If you have not taken your medication then please enter **0**.

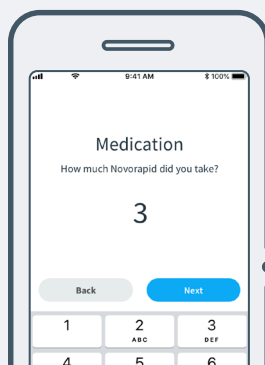

### Step 7

If you want to tell your diabetes care team something about your reading then please type this information into the **New comment** screen and then submit.

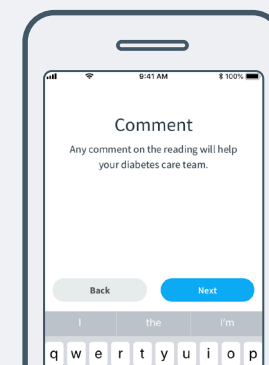

### Step 8

Your reading will then appear on your diary and graph screens.