

# CORE

## Quick Start Guide

Download the Eko app from the Apple App Store® or Google Play Store®.

Eko

Turn on bluetooth on your phone or tablet, launch the app, and turn on the CORE.

### In the Box

- Eko CORE with a cardiology stethoscope
- Removable bell and pediatric diaphragm
- 3 sets of soft-seal eartips
- USB charging cable
- Getting started guide

### Volume

Use the (+) and (-) buttons to adjust volume. There are 7 volume steps.

### Turn On and Off

When the CORE is off, it functions as an unamplified stethoscope.

To turn the CORE on, press the power toggle in. The light will begin flashing. You will now hear amplified and filtered sound in the eartips.

### Telemedicine Livestream Subscription

The Eko App includes optional telemedicine livestream features that require a software subscription. You can setup a subscription online at:

<https://dashboard.ekohealth.com>



### Active Noise Cancelling

The CORE includes active noise cancelling to remove ambient noise from the surrounding environment. Active noise cancelling can be turned off or on through the Eko App.

### Notifications

 CORE is on & seeking device	 CORE is off & charging	 CORE is low on battery
 CORE is on & connected	 CORE is fully charged	 CORE is recording


### Recording

When connected to the Eko app, press the (o) button to begin recording.



### Placement

The CORE is designed to be used in all traditional auscultation positions for the heart, lungs, bowels, and arteries.

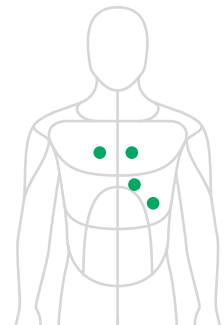

### Charging

Connect the micro USB cable to the CORE and use a UL-certified USB wall charger. The CORE will take about 2 hours to fully charge. The light will turn green when it is fully charged.



### User Manual and Support

Review the user manual before using your Eko CORE. To view the manual visit: [ekohealth.com/getstarted](http://ekohealth.com/getstarted)

For support, please email: [contact@ekohealth.com](mailto:contact@ekohealth.com)

TM and © Eko Devices Inc.

Apple App Store is a trademark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC.

LBL 056 Rev A