

Active Jogging Shoe

A hoof shoe with a wide range of suitability for rehabilitation, terrain riding, and jumping.



- EASY TO PUT ON
- SOFT PADDING
- COMFORTABLE
- FLEXIBLE AND DURABLE SOLE TECHNOLOGY
- WEAR RESISTANT VELCRO
- ADJUSTABLE FRONT COVER WITH SMOOTH INNER SIDE
- SMART LOCK FOR EXTRA FASTENING SECURITY



Active Jogging Shoe

The key features for the flexible and elastic Jogging shoe

- 1) Safeguards the horse's natural balance
- 2) Safeguards the horse's natural **stability**
- 3) Ensures the horse's need for **cushioning**
- 4) Promotes a more consistent surface contact
- 5) Promotes a more **natural rollover motion**

(utilize entire area)

(utilize entire area)

(natural hoof movement, absorb shock & vibrations) (adapt to the hoof-ground interface, customized grip) (natural hoof movement, customized grip)

Size	Size Range Length	Size Range Width	Break over - Point	Weight
7	6.6-7.5cm (2 5/8 - 2 15/16")	up to 7cm (2 3/4")	7 mm	145 gr
8	7.6-8.5cm (3 - 3 3/8")	up to 8cm (3 1/8")	7 mm	165 gr
9	8.6-9.5cm (3 3/8 - 3 3/4")	up to 9cm (3 9/16")	7 mm	205 gr
10	9.6-10.5cm (3 3/4 - 4 1/8")	up to 10cm (3 15/16")	10 mm	265 gr
11	10.6-11.5cm (4 3/16 - 4 1/2")	up to 11 cm (4 5/16")	10 mm	310 gr
12	11.6-12.5cm (4 9/16 - 4 15/16")	up to 12 cm (4 3/4")	10 mm	350 gr
13	12.6-13.5cm (4 15/16 - 5 5/16")	up to 13cm (5 1/8")	10 mm	390 gr
14	13.6-14.5cm (5 3/8 - 5 11/16")	up to 14cm (5 1/2")	10 mm	490 gr
15	14.6-15.5cm (5 3/4 - 6 1/8")	up to 15 cm (5 15/16")	10 mm	570 gr
16	15.6-16.5cm (6 1/8 - 6 1/2")	up to 16cm (6 5/16")	10 mm	610 gr

Dampening Factor		Terrain	
Sole	Medium +	Hard to medium soft, plus ice (when Stud Kit All Terrain installed)	
Sole with pad	Soft +		

www.eqfusion.com



Active Jogging Shoe

QUICK STEP- installation



1. Open the shoe wide





2. Place the shoe on the hoof. Make sure that the toe is all the way in front of the shoe, and that the shoe is centered in the back.



3. Put the Velcro together. Check that both Velcro's are aligned on top of each other.



4. Alternatively, cross the Velcro downwards to get a better fit around the pastern.



5. Pull the strap firmly with a hoof pick and secure. This will pull the shoe more tightly around the hoof.



6. Close front cover and secure the smart lock on the side in the hole which fits your horse.



7. Ready to run!

(^{(IIII})) equinefusion

www.eqfusion.com

equinefusion (f)



www.eqfusion.com | Regimentveien 128, 4045 Hafrsfjord Norway +47 911 50 989 | contact@eqfusion.com



Follow us on social <u>media</u>

Please share your Equine Fusion experience with us. Post on our Facebook page or use **#equinefusion** on Instagram.





@EQFUSION