

Our cleaning regime for the Rondoflex 360

Note: We use only 27 micron powder, in the corresponding blue container

1. Remove blue powder container after use, and immediately cover with its lid to prevent any moisture entering
2. Half fill the second unused/spare (50 Micron) container with water and attach to the Rondoflex
3. Connect the Rondoflex to the coupling and run it for 30 seconds
4. Remove the blue container, rinse with water, half fill, re-attach it, and run again for another 30 seconds. Repeat a third time.
5. Use triplex syringe (clean, dry air) to blow air through the internal line of the Rondoflex to remove any moisture.
6. The components can then be sterilized.
7. Be sure to blow air through the Rondoflex after sterilization to ensure it is dry, before it is stored or before new, dry powder is re-attached to the Rondoflex for its next use.

