

Resources

LEARNING AND EDUCATION PLATFORM

Animation

For more guidance on how to
use Fetal Pillow please visit:

safeob.com/animation

PLAY

Online Training

To use the 15-minute online
training tool visit:

[sos.elearningplatform.net/
dashboard/signup](http://sos.elearningplatform.net/dashboard/signup)

REGISTER

Download the App




Scan the QR code using your
mobile device

1. Insertion




Fold the baseplate along the short axis
ensuring balloon is on the inside.
Tube attachment must be at superior
end (pointing upwards)
Insert into the vagina

2. Placement

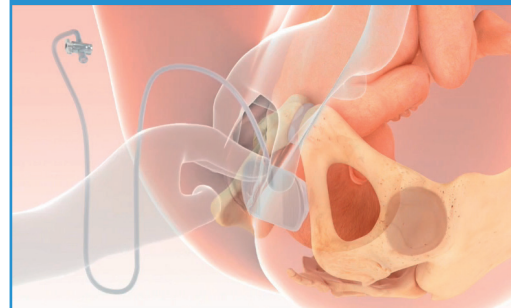

Ensure balloon surface is in contact with
the fetal head
Push device posteriorly, towards the coccyx
Placement is similar to a posterior
Ventouse / vacuum cup

3. Legs Flat

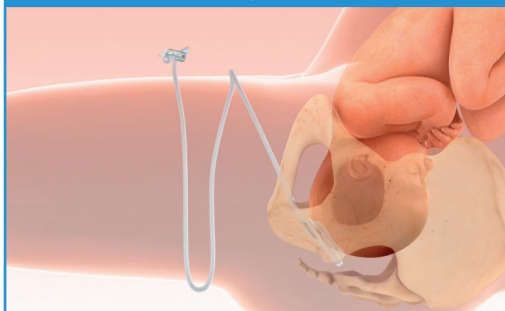

Place patients legs flat on the table
before inflation

4. Inflation



Use the syringe to inflate using 180ml of
sterile saline
Close tap after filling to prevent fluid
from leaking

5. Deliver baby

Perform Caesarean section using
standard technique

6. Deflation & removal

Once baby has been delivered open the
tap to drain the fluid
Remove the device at the end of the
procedure