

Product information

REF 730-5



REF 730-5

TRACOE smart Cuff Manager


- Packaging unit:
- ◆ 5 units
 - ◆ sterile, individually packaged

The TRACOE smart Cuff Manager monitors and regulates the internal pressure of High Volume Low Pressure cuffs of tracheostomy tubes as well as endotracheal tubes.

The pressure is maintained between 20cmH₂O and 30cmH₂O. Visual verification of the cuff pressure is indicated by the inflation level of the blue buffer ball of the TRACOE smart, which should be inflated between 2/3 and 3/4. Additional checks of the cuff pressure may be performed with a cuff pressure manometer (e. g. TRACOE cpm REF 720). Is the cuff pressure too low, the TRACOE smart Cuff Manager refills the cuff quickly to its appropriate pressure.

Short high pressure peaks e. g. due to coughing will be tolerated by the integrated damping function which inhibits that air escapes instantly from the cuff. The solid shell of the TRACOE smart protects the highly elastic blue balloon from unwanted compression.

TRACOE smart is a latex- and DEHP-free, single patient product with a usage time of 29 days after opening.

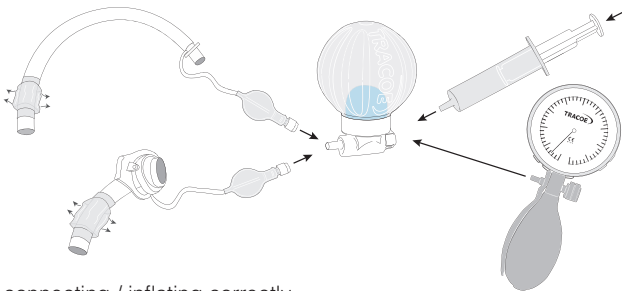

connecting / inflating correctly

Connecting tube

Following connecting tube for TRACOE smart Cuff Manager can be ordered separately:

- ◆ TRACOE technic Connection Tube REF 702
1 m length with check valve