

Product information

REF 103-A-O



REF 103-A-O



TRACOE *comfort* Tracheostomy Tube

fenestrated with oxygen supply port attached to the inner cannula


- Including:
- ◆ inner cannula, fenestrated, with slide-on speaking valve type A (silver valve holder) and oxygen supply port
 - ◆ inner cannula with 15mm connector
 - ◆ inner cannula with low profile
 - ◆ neck strap

For tracheostomised patients with fully or partially intact larynx, for speaking. Made upon order.

Technical Data

REF 101, REF 102, REF 102-D, REF 103, REF 103-O, REF 103-A, REF 103-A-O, REF 104, REF 104-O, REF 104-A, REF 104-A-O, REF 105, REF 105-D, REF 107, REF 108

REF 114 and REF 115 *in sizes 07 – 11*

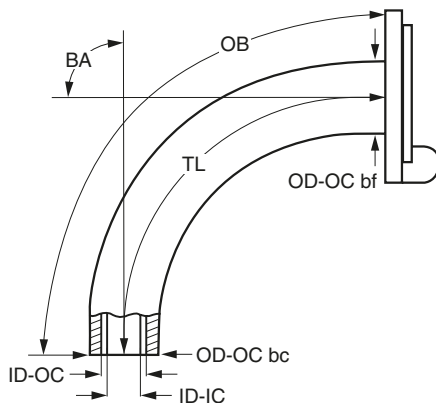

Size	ID-OC mm	OD-OC bc mm	OD-OC bf mm	ID-IC mm	TL mm	OB mm	BA e°
03	3.5	4.8	6.4	2.6	42	46	90°
04	4.0	5.6	7.3	3.2	50	55	90°
05	5.0	6.7	8.7	4.0	52	57	90°
06	6.0	7.6	10.0	4.8	54	60	90°
07	7.0	8.8	10.5	5.8	57	65	90°
08	8.0	10.0	11.3	6.8	61	70	90°
09	9.0	11.4	12.6	7.6	65	75	90°
10	10.0	12.4	13.7	8.7	74	85	90°
11	11.0	13.1	15.0	9.7	76	87	90°
12	12.0	14.2	16.2	10.8	78	90	90°
13	13.0	15.4	17.2	11.8	87	100	90°
14	14.0	16.6	18.2	12.6	91	105	90°


Tubes of size 03, 04, 13 and 14 are made to order. Delivery time is approx. 10 working days.

ID-OC: inside diameter (clear width) at bottom of outer cannula; **OD-OC bc:** outside diameter at bottom of outer cannula; **OD-OC bf:** outside diameter of outer cannula behind the flange; **ID-IC:** inside diameter at bottom of inner cannula; **TL:** length along centre line from start of neck flange to bottom of tube; **OB:** length along outer bend (greatest length); **BA:** bending angle

To order, specify **REF + size**, e.g. **REF 103-08**

Product information

REF 103-A-O

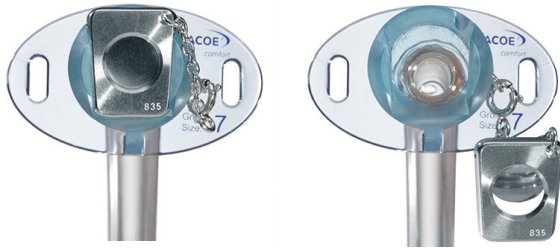

- ◆ TRACOE *comfort* Speaking Valve, type A

TRACOE *comfort* tubes with type A speaking valves have a silver valve holder that is attached to the inner cannula with a chain. When necessary, it can be pulled off the inner cannula, allowing free access to the lumen, e.g. for suctioning off tracheal secretions.

Scope of Supply of TRACOE *comfort* Tubes



All TRACOE *comfort* tubes are supplied with a wide adjustable neck strap and a product info card with two detachable labels.

Accessoires

- ◆ TRACOE humid assist I REF 640-CT
Heat & Moisture Exchanger with foam filter for attachment to any tube with 15 mm connector
- ◆ TRACOE humid assist III REF 641
Heat & Moisture Exchanger with paper filter, integrated oxygen supply port, turnable through 360°, for attachment to any tube with 15 mm connector
- ◆ TRACOE humid assist VI REF 642
Heat & Moisture Exchanger with foam filter, integrated oxygen supply port and pressure control valve for expectoration of larger quantities of secretion, for attachment to any tube with 15 mm connector

Cleaning products

- ◆ TRACOE care Liquid Paraffin REF 902
lets inner cannulas be easily inserted and removed, assures best possible fit and good quality of speech
- ◆ TRACOE tube clean Cleaning Kit REF 930-A/-B
with cleaning powder REF 932
- ◆ TRACOE tube clean Cleaning Kit REF 931-A/-B
with concentrated cleaning liquid REF 933

Product Variants

Product variants can be made in order to optimise the functionality and wearing comfort and also for patients with complex anatomical and physiological conditions. The relevant ordering forms are available at www.tracoe.com/en/downloads.