TRACOE medical GmbH
Reichelsheimer Straße 1 / 3, 55268 Nieder-Olm / Germany
Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200
info@tracoe.com, www.tracoe.com

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EN

Instructions for Use for TRACOE® 22 mm Adapter REF 622

These instructions for use contain information and instructions for physicians, nursing staff, patients and relatives who act as carers. They exclusively apply to the 22 mm adapter. Please read the notes before using the product for the first time and retain the instructions for use for future reference.

The initial application of the adapter must be carried out by a physician or by trained qualified personnel. The users must be trained by qualified personnel in handling the product, and especially in the functioning and handling of speaking valves.

Packaging Contents: Each REF 622 package contains a TRACOE® 22 mm adapter each.

1. Intended Purpose & Use

This adapter is used for plugging a heat and moisture exchanger or a speaking valve with male 22 mm attachment onto tracheostomy tubes with 15 mm connectors (see illustration above).

Product for Single-Patient Use: This medical product is designed for use with only one patient. Multiple use on one and the same patient is permitted.

2. Indication

The TRACOE® 22 mm adapter is used for **tracheostomised and laryngectomised** patients who are using a tracheostomy tube with a 15 mm connector. Heat and moisture exchangers as well as speaking valves, which are designed for 22 mm adapters, can be inserted in the 22 mm adapter.

3. Contraindications

CAUTION: Tubes with a speaking valve must not be used for laryngectomised patients without a voice prosthesis, because these patients cannot exhale the inhaled air, and there is a risk of asphyxiation.

The adapter is not suitable for ventilation because it may detach from the 15 mm connector.

4. Handling the 22 mm Adapter

In general, the patient should plug the 22 mm adapter into the 15 mm connector and remove the adapter from the connector himself or herself. The reason is that the patient knows best how much force to apply on the tracheostomy tube.

CAUTION: The 15 mm connector of the tracheostomy tube as well as the 22 mm adapter and the attachment that is connected to it should be free from all lubricants such as oil or lubricating gel. **CAUTION:** When changing the patient's clothes or position, make sure that the adapter doesn't get caught or become detached.

At first, insert the 22 mm attachment (heat and moisture exchanger or speaking valve) into the 22 mm adapter until it latches. Then, push the adapter onto the 15 mm connector. During this process, tightly hold both the adapter and the attachment in order to avoid the accidental dislocation of the attachment (heat and moisture exchanger or speaking valve).

For further information on the use of the product, please refer to the instructions for use of the respective products.

In order to remove the 22 mm adapter, please proceed in reverse order: First, pull the 22 mm adapter together with the attachment off the 15 mm connector of the tracheostomy tube while making sure that the tube is fixed to the neck flange. Then separate the attachment from the adapter.

5. Cleaning and Care

CAUTION: Make sure that the air-conducting parts of the 22 mm adapter as well as the attachments (heat and moisture exchanger or speaking valve) and the tracheostomy tube are free from secretion or crust formation in order to avoid any unintended blockage of the airflow.

The cleaning intervals heavily depend on the main illness of the patient as well as the secretion volume and consistency, any tendency to crust forming and the patient's individual needs.

Separate the adapter from all other parts before cleaning it. We recommend that you use the TRACOE cleaning products (REF 930-A or -B or optionally REF 931-A or -B) for cleaning the 22 mm adapter. The individual cleaning steps are detailed in the instructions for use of the respective products.

In many cases, the adapter is used for out-of-hospital applications and domestic care. In such cases, and due to the germ range, which is different from that in hospitals, under normal circumstances simplified cleaning using a commonly available, domestic, mild and pH-neutral agent is possible.

CAUTION: The adapter must not be cleaned with aggressive domestic cleaning agents, cleaning agents for dentures or agents with a high alcohol content because these agents can impair the functioning of the product.

The adapter must be checked for possible damage after each cleaning and before its reuse. Even adapters that show only minor damage must be replaced.

Disinfection: The use of unsuitable disinfectants can result in damage to the product. Please contact TRACOE medical for a list of tested disinfectants (www.tracoe.com).

After the adapter has been cleaned or disinfected, it must be rinsed with tap water of drinking water quality or with distilled water and then dried.

CAUTION: Do not use any disinfectants, which release chloride, or agents, which contain strong alkaline solutions or phenol derivatives.

You may thermally disinfect the adapter with a mild agent with a neutral pH for 5 minutes at temperatures of up to 65 °C without

using any chemical additives. This process may result in an opaque staining, but the function of the product is not impaired.

6. Useful Life

The useful life heavily depends on the patient's illness as well as the secretion and care of the 22 mm adapter. The adapters can be used for up to 6 months. After this time, the adapter must be renewed. Any use beyond its useful life will result in the warranty becoming void. This means that the manufacturer will no longer assume any liability for damage that is caused to the products and for consequential damage.

7. Disposal

These products may only be disposed of in accordance with the valid national regulations governing waste materials.

8. Returns

Returns of used products are accepted only if accompanied by a completed decontamination certificate. You can obtain the required form either directly from TRACOE medical or from the website (www.tracoe.com).

9. General Terms and Conditions

The sale, delivery and return of all TRACOE® products are carried out exclusively according to the General Terms and Conditions which can be obtained directly from TRACOE medical GmbH or from the website (www.tracoe.com).

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