

Product information

REF 330



REF 330 

TRACOE experc Set *twist plus*
including **REF 311-P**
with low-pressure cuff and
minimally traumatic insertion system

The systems for *twist plus* consist of two parts: the white guiding catheter with silicone sleeve and the green inserter. The silicone sleeve bridges the gap in diameter between the end of the tube and the inserter. The *twist plus* tracheostomy tube is supplied premounted on the minimally traumatic insertion system, with the inner cannula already in place. The insertion system stabilizes and facilitates guiding the tube. The inserter fits the tube precisely and has a stop at the 15 mm connector. Once the tube is in the trachea, the colored inserter, the white guiding catheter with the silicone sleeve, and the Seldinger guide wire are removed together as a single unit. The silicone sleeve flaps back, and the unit can be removed.

The Benefits

Dilatation is performed using the Ciaglia one-step technique. The minimally traumatic inserter smooths over the gap in diameter between the end of the tube and the inserter. The stop prevents the inserter from being inserted too far into the tube, which reduces the risk of injury. The separately available P-tubes can also be used for reinsertion and/or for tube change – in combination with a Seldinger guide wire, if necessary.

experc Sets *twist plus*

The TRACOE *twist plus* P-tubes REF 311-P, REF 312-P, REF 316-P and REF 888-316-P as well as the TRACOE experc Dilatation Set REF 520 are also available separately.

You can find details on the tube in the corresponding product information.

Accessories

Spare Inner Cannulas

- ◆ TRACOE *twist plus* Spare Inner Cannulas REF 521, unfenestrated, 3 units, supplied in individual sterile packages
- ◆ TRACOE *twist plus* Spare Inner Cannulas REF 521-X bulk pack, unfenestrated, 10 units, supplied in individual sterile packages
- ◆ TRACOE *twist plus* Spare Inner Cannulas REF 521-XXX bulk pack, unfenestrated, 30 units, supplied in individual sterile packages

Heat/Moisture Exchangers, HME

- ◆ TRACOE humid assist I REF 640-CT
- ◆ TRACOE humid assist III REF 641
- ◆ TRACOE humid assist VI REF 642

For the Cuff Pressure Management

- ◆ TRACOE smart Cuff Manager REF 730-5
- ◆ TRACOE cuff pressure monitor (cpm) REF 720
- ◆ TRACOE cuff pressure monitor sensitive (cpms) REF 721