

Product information

REF 320



REF 320

TRACOE experc Set *twist*
including **REF 301-P**
with low-pressure cuff and
minimally traumatic inserter

At the tip of the *twist* inserter, there is a folding silicone sleeve that bridges the gap in diameter between the inserter and the end of the tube. The minimally traumatic inserter is adapted to the respective tube and its size with a stopper at the proximal end of the tube. This minimizes slipping of the tube. Once the tube is in the trachea, the inserter is pulled out of the tube. The silicone sleeve flaps back to facilitate removal of the inserter.

The Benefits

Dilatation is performed using the Ciaglia one-step technique. The minimally traumatic inserter smooths over the gap in diameter between the end of the tube and the inserter. The stop prevents the inserter from being inserted too far into the tube, which reduces the risk of injury. The separately available P-tubes can also be used for reinsertion and/or for tube change – in combination with a Seldinger guide wire, if necessary.

experc Sets *twist*

The TRACOE *twist* P-tubes REF 301-P, REF 302-P, REF 306-P and REF 888-306-P as well as the TRACOE experc Dilation Set REF 520 are also available separately.

You can find details on the tube in the corresponding product information.

Accessories

Spare Inner Cannulas

- ◆ TRACOE *twist* Spare Inner Cannulas REF 501, unfenestrated, 3 units, supplied in individual sterile packages
- ◆ TRACOE *twist* Spare Inner Cannulas REF 501-X bulk pack, unfenestrated, 10 units, supplied in individual sterile packages
- ◆ TRACOE *twist* Spare Inner Cannulas REF 501-XXX bulk pack, unfenestrated, 30 units, supplied in individual sterile packages

Heat/Moisture Exchangers, HME

- ◆ TRACOE humid assist I REF 640-CT
- ◆ TRACOE humid assist III REF 641
- ◆ TRACOE humid assist VI REF 642

For the Cuff Pressure Management

- ◆ TRACOE smart Cuff Manager REF 730-5
- ◆ TRACOE cuff pressure monitor (cpm) REF 720
- ◆ TRACOE cuff pressure monitor sensitive (cpms) REF 721