



Reducing the burden of pressure injuries


Optimising clinical outcomes
and patient experience.

Helping you get CLOSER TO ZERO[®] pressure injury incidence

 **smith&nephew**

ALLEVYN[®] LIFE

Foam Dressing

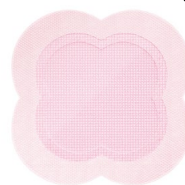

Supporting healthcare professionals

The strain on healthcare resources

On average, patients with hospital acquired pressure injuries (HAPIs) spend another 3.7 days in hospital compared to those without HAPIs¹


Patients with HAPIs typically cost \$8014¹ more than those who did not acquire pressure injuries (PIs). This puts an added strain on the availability of hospital beds and the families who have to support loved ones through longer hospital stays.¹

Published international guidelines recommend the use of foam dressings prophylactically, to help protect bony prominences (like heels and sacrum) and areas of tissue under medical devices that are frequently subjected to pressure, friction and shear.²


Dressings should be used alongside standard preventive care and allow for frequent skin inspections.²



\$11 463 AUD**
extra cost^{*1}

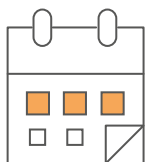

3.7 days more
in hospital^{*1}

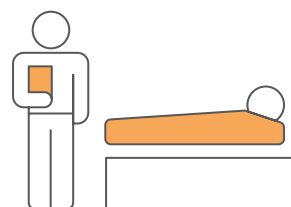

Greater odds of re-admission
within 30-180 days^{*1}

*compared to patients without HAPIs

ALLEVYN[◊] LIFE dressings help makes a difference

ALLEVYN[◊] LIFE dressings improve outcomes and save money

In an independent randomised controlled trial, ALLEVYN[◊] LIFE dressings reduced sacral pressure injuries by 71% in elderly hip fracture patients³ and showed a cost saving of 69% per patient – compared to using standard preventive care alone.⁴

Reduce sacral PIs³ by 71%



Cost saving of 69%⁴



**6/5/19 exchange rate

ALLEVYN[◇] LIFE dressings take the pressure off

An innovative five layer silicone adhesive foam dressing

- ALLEVYN LIFE is an all-in-one dressing suitable for wound management and Pressure Injury PI prevention. Its innovative design and construction redistributes pressure (*in vitro*)⁵
- ALLEVYN LIFE dressings stay in place for up to 7 days^{6*}
- To help facilitate regular skin inspections, ALLEVYN LIFE allows lifting and repositioning without losing its adherent properties⁷⁻⁹



*as demonstrated in volunteers

ALLEVYN LIFE dressings suit individual needs

The ALLEVYN LIFE range can suit each patient's individual needs

With a wide range of shapes and sizes, ALLEVYN LIFE dressings are designed to conform to the body, helping to enable patient comfort⁷ during wear.



ALLEVYN LIFE



ALLEVYN LIFE Heel



ALLEVYN LIFE Sacrum

ALLEVYN[®] LIFE ordering codes

Product code	Product description	Dimensions	Carton quantity
66801067	ALLEVYN LIFE Small	10.3cm x 10.3cm	10
66801068	ALLEVYN LIFE Medium	12.9cm x 12.9cm	10
66801069	ALLEVYN LIFE Large	15.4cm x 15.4cm	10
66801070	ALLEVYN LIFE Extra Large	21cm x 21cm	10
66801304	ALLEVYN LIFE Heel	25cm x 25.2cm	5
66801306	ALLEVYN LIFE Sacrum	17.2cm x 17.5cm	10
66801307	ALLEVYN LIFE Sacrum	21.6cm x 23cm	10

Smith & Nephew Pty Ltd
85 Waterloo Rd
North Ryde NSW 2113
Australia
T +61 2 9857 3999
F +61 2 9857 3900

Smith & Nephew Ltd
Unit A 36 Hillside Rd
Wairau Valley Auckland 0627
New Zealand
T +64 9 820 2840
F +64 9 820 2841

www.smith-nephew.com/australia
www.smith-nephew.com/new-zealand

®Trademark of Smith & Nephew
All Trademarks acknowledged
AWC-15548_AU V1 | SN12345 (04/19)

Supporting healthcare professionals for over 150 years

References

1. Dreyfus J, Gayle J, Trueman P, Delhougne G, Siddiqui A. Assessment of Risk Factors Associated With Hospital-Acquired Pressure Injuries and Impact on Health Care Utilization and Cost Outcomes in US Hospitals. *Am J Med Qual.* 2018;33(4):348-358 2. National Pressure Injury Advisory Panel, European Pressure Injury Advisory Panel and Pan Pacific Pressure Injury Alliance. Prevention and Treatment of Pressure Injuries: Quick Reference Guide. Emily Haesler (Ed.). Cambridge Media: Osborne Park, Australia; 2014 p.45 3. Forni C, D'Alessandro F, Gallerani P, et al. Effectiveness of using a new poly-urethane foam multi-layer dressing in the sacral area to prevent the onset of pressure injury in the elderly with hip fractures: A pragmatic randomised con-trolled trial. *Int Wound J.* 2018;15(3):383-390 4. Forni C, Searle R. Economic evaluation of the use of a multi-layer polyurethane foam dressing for the prevention of pressure injuries in elderly patients with hip fractures. Poster presented at EPUAP annual meeting, September 2018 5. Data on File Report DS/15/217/R – October 2015, L. Daubney. Pressure Transmission Testing 6. ALLEVYN[®] LIFE vs Mepilex[™] Border – Product Retention Performance Testing. Data on File report HVT080 September 2018 7. Rossington A. Product Performance of Next Generation ALLEVYN Life (HVT080) 2016. GMCA-DOF/08 8. Clarke B. Positive patient outcomes: The use of a new silicone adhesive foam dressing for pressure injury prevention and treatment. Paper presented at: CAET2013 9. Lisco C. Evaluation of a new silicone gel-adhesive hydrocellular foam dressing as part of a pressure injury prevention plan for ICU patients. Paper presented at: WOCN2013 10. Data on File Report DS/15/025/R- May 2016, L. Daubney. Physical testing ALLEVYN LIFE Gen2 11. Data on File Report DS/16/061R.- Feb 2016, E. Mumby. Subjective comparison of masking ability of ALLEVYN LIFE Gen2 Data on File Report DS/14/303/R.- April 2016, K.Earl. Wound 12. Data on File Report DS/14/303/R.- April 2016, K.Earl. Wound model testing of ALLEVYN LIFE Gen2 using horse serum.