
















## Toolbar Buttons

	Study/Patient Search Availability is dependent upon site's configuration.
	Navigator
	Series Selector
	Layout Grid View_H (2H) and View_V (2V)
	Window Level Presets <ul style="list-style-type: none"> <li>List of site defined presets</li> </ul>
	Flip/Rotate <ul style="list-style-type: none"> <li>Flip vertically</li> <li>Flip horizontally</li> <li>Rotate right</li> </ul>
	Measure <ul style="list-style-type: none"> <li>Distance</li> <li>Freehand (ROI)</li> </ul>
	Image Overlay Toggle
	Cross Reference
	Presentation States <ul style="list-style-type: none"> <li>List of Presentation States</li> </ul>
	Cine
	External Applications
	Ambient Light Test
	Reports <ul style="list-style-type: none"> <li>View reports and exam notes</li> </ul>

## Tips

Grayed out tools are either not valid for the study or not configured for your site.

Click **Help**  to access the detailed operators' manual.

To **logout**, click the logout button .

# Centricity™ Universal Viewer Zero Footprint

## iPad® Job Aid

Zero Footprint is not for diagnostic use.

iPad is a registered trademark of Apple, Inc.,

General Electric reserves the right to make changes in specifications and features, or discontinue the product or service described at any time, without notice or obligation. This does not constitute a representation or warranty or documentation regarding the product featured. All illustrations or examples are provided for informational or reference purposes and/or as fictional examples only. Your product features and configuration may be different than those shown. Information contained herein is proprietary to GE. No part of this publication may be reproduced for any purpose without written permission of GE.

GE Healthcare, 540 W. Northwest Highway, Barrington, IL 60010

© 2016 General Electric Company. All rights reserved.

GE, the GE Monogram and Centricity are trademarks of General Electric Company.

Software Version 6.0.X

Document Revision 2

Document Number DOC1444293




## Centricity Universal Viewer Zero Footprint on iPad

Universal Viewer Zero Footprint (ZFP) can be accessed via an iPad. This job aid provides information for image display and manipulation on the iPad. ZFP via an iPad, as with the ZFP client, is not for diagnostic use. It is a review-only offering.

### Ambient Light Test

The first time you launch ZFP on the iPad or upon moving to a new environmental lighted condition you should perform the following ambient light test

1. On the toolbar, tap the **Ambient Light Test** icon .
2. Review the light test information.
3. Tap **Perform Light Test**.
4. When the TG18-CT pattern displays, verify that you can see an image of a half-moon in each of the 16 luminance regions.
5. After you are successful, either:
  - Swipe on the page from right to left.
  - Tap **TG18-MP** in the top right corner of the screen.
6. When the TC18-MP pattern displays, verify that you can clearly distinguish between the successive grayscale bands.
7. After you are successful, tap **Done**.  
A confirmation messages displays requesting you confirm the test results.
8. If you select **Yes**, the ZFP client iPad version launches.  
If you select **No**, the Ambient Light Test application instructs you to find more suitable ambient lighting.

## iPad Finger Gestures

	<b>Adjust Window Level and Width</b> <ul style="list-style-type: none"> <li>• Change the level: Tap the viewport and swipe a single finger up and down vertically.</li> <li>• Change the width: Tap the viewport and swipe a single finger left and right horizontally.</li> </ul>
	<b>Pan</b> <ul style="list-style-type: none"> <li>• Tap image with <b>two fingers</b> and <b>drag</b> to desired position.</li> </ul>
	<b>Zoom</b> <ul style="list-style-type: none"> <li>• <b>Pinch in</b> and <b>pinch out</b> using two fingers.</li> </ul>
	<b>Activate Series Slider/Slicer Bar for Image Navigation</b> <ul style="list-style-type: none"> <li>• Tap near the bottom of the screen to activate the Series Slider / slider bar for Image Navigation. You can advance to a particular image based on the relative location on the bar. Images may be skipped.</li> </ul>
	<b>Image Navigation</b> <ul style="list-style-type: none"> <li>• Drag <b>single finger</b> on the Series Slider / slider bar to scroll through the images in the series.</li> </ul>
	<b>Navigating to the Next Series</b> <ul style="list-style-type: none"> <li>• To navigate to the first image of the next series, use a <b>single finger</b> in the viewport to <b>swipe</b> from <i>right to left</i>.</li> </ul>

	<b>Navigating to the Next Image in a Series</b> <ul style="list-style-type: none"> <li>• Use a <b>single finger</b> and <b>drag</b> in a <i>vertical</i> direction.</li> <li>• <b>Swipe top to bottom</b> to go to the next image.</li> <li>• <b>Swipe bottom to top</b> to go to the previous image.</li> <li>• If you reach the end of the screen, continue to <b>hold a single finger</b> on the screen to navigate to the next image within a series.</li> </ul> <p>BOLD GESTURES IN THE SENTENCES ABOVE AS APPROPRIATE AND REMOVE EXTRA SPACE</p>
-------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------