

USER'S MANUAL




aura
Pulse




Turning on your Aura Pulse

1. Align the charging port pin with the gold dots on the back of the Aura Pulse.
2. Put the Aura Pulse into the charging port gently. Push until you hear and feel a snap.
3. Plug the USB cable into a power source.

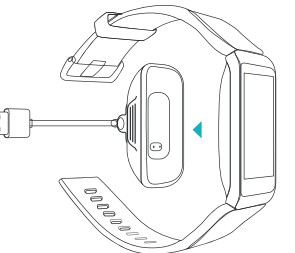


Removing the wristband

To remove the wristband, find the Quick Release pin at the bottom of the Aura Pulse. Press the pin inward, and pull to remove the wristband gently.

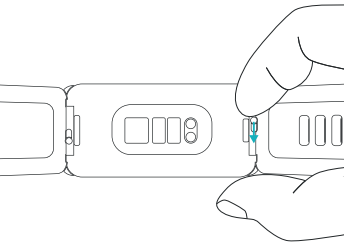


Installing the wristband

To install the wristband, align one side of the wristband with the hole of the Aura Pulse, push the exposed pin in, and press the wristband into place.

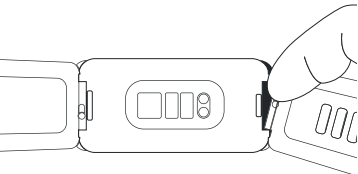

Activation and syncing

1. Download the AuraTrack™ app on your compatible iOS or Android device.
2. Enable Bluetooth on your iOS or Android device.
3. Turn your Aura Pulse on.
4. Pair your device with Aura Pulse

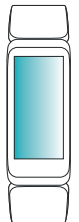

Gestures for using the touchscreen



Swipe up or down on the touchscreen to scroll through all features.



Slide left for the today view, and slide right for the day/night mode.



Slide right when not on the home page to return to the previous page.



Cover the screen* fully for 3 seconds to return to the home page.

*Covering the screen to return to the home page doesn't work in while taking SPO2 or Heart rate measurements, or while using Activity tracking.



Preinstalled apps



Heart rate
Continuous heart rate monitoring, with measurements for active and resting heart rate



Blood oxygen
Measure your SPO2 levels anytime, anywhere



Today
View your daily step, heart rate, and sleep data all in one place



Alarm clock
Set alarms and reminders on the Aura Pulse, and wake up to a gentle buzz



Settings
Change to a beautiful selection of watch faces, and change Aura Pulse settings

What's in the box



Health Tracker



Charging cable



Wristband



User manual

Product Specifications

Name:	Aura Pulse
Model:	LS456-B
Connectivity:	Bluetooth 5.0
Display screen:	1.05” TFT LCD
Resolution:	120 × 240 px
Wristband materials:	Solid silica gel
Battery capacity:	130mAh
Operating voltage:	5V
Operating temperature:	-10°C ~ 40°C
Weight:	0.8oz (22g)
Waterproof rating:	IP68
Battery life:	5 ~ 7 days
Device requirements:	iOS 11.0 or above, Android 6.0 or above, and Bluetooth 4.0 or above

Medical Disclaimer:
Warning: This product is not a medical device.
The Aura Pulse Health Tracker and related applications shall not be used to diagnose, treat or prevent any disease or medical conditions. Before adjusting your exercise, sleep, or diet plan, consult a medical professional.

Important safety instructions:

1. If you have any existing medical conditions which may be affected by the use of this product, please consult your doctor.
2. Prolonged use of the Aura Pulse may cause skin irritation in some users. Please consult your doctor if you experience any redness, swelling, itching, or other allergy symptoms.
3. Do not use or look at Aura Pulse while driving or in situations where concentration is required.
4. This item is not a toy and is not intended for use by children. It contains small parts that could pose a choking hazard if swallowed. Keep out of reach of children.
5. The performance of this product may be affected by external factors. Therefore, the data provided by this product should be used for reference only.
6. This product is intended to record your heart rate function. If worn for long periods of time, it may cause skin discomfort or allergy

symptoms in some users or pose a risk to users with certain medical conditions. Please stop using this product or consult your doctor before using if you have any of the following conditions:


- Epilepsy or light sensitivity
- Poor blood circulation or bruising
- Heart disease or other medical conditions
- If you are taking any photosensitive drugs

Wear and Maintenance Tips:

1. Be careful to adjust the tightness of the Aura Pulse so that it is not too tight to ensure smooth blood circulation.
2. Wear the Aura Pulse fairly tightly to your skin during workouts or other physical activity.
3. Do not apply excessive skincare products to the wrist with the Aura Pulse.
4. Remove the Aura Pulse immediately if you feel any temperature changes or excessive heat.
5. Take the Aura Pulse off from time to time to clean it and give your skin a chance to breathe. You can use this time to recharge the Aura Pulse as well.
6. Regularly clean and dry the Aura Pulse with a clean, wet cloth, paying particular attention to areas that contact with your skin.

Aura Pulse is an electronic device. If it is not handled or disposed of properly, it could damage the environment or cause injury.

Do not disassemble this product. There are no serviceable parts inside.
Do not replace the built-in battery; this should only be done by a qualified technician.
Do not clean this product with corrosive detergent, which could damage the device.
Do not charge this product in wet conditions.
Do not charge this product before it is completely dry.
Do not expose this product to direct sunlight for extended periods of time.
Do not use this product in a sauna or steam room.
Do not place this product in extremely high or low temperatures.
Do not use this product if the screen or casing is damaged.
Do not put this product in the dishwasher, washing machine, or dryer.
Do not put this product in fire, as the battery may explode.
Use the USB port of a computer or mobile phone charger to charge this product.
If the temperature of this product rises too high during charging, stop charging immediately.

 FCC ID: OU9LS456-B

This device complies with part 15 of the FCC rules . Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.

- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help
- This device and its antenna(s) must not be co-located or operating in conjunction with any other antenna or transmitter.

FCC Radiation Exposure Statement
This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment.
This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Frequently Asked Questions:

• **What can I do if my Aura Pulse fails to connect to the AuraTrack app?**


1. First, check to see if the Aura Pulse's battery is low.
2. Next, check to see if multiple mobile phone devices are trying to connect to the Aura Pulse. For example, if you have used Aura Pulse with an iOS device and are now pairing Aura Pulse to an Android device, you will need to unpair Aura Pulse from your iOS device. You can do that by opening the Settings app on your iOS device, open Bluetooth settings, locate your Aura Pulse, and tap Forget.
3. If you still cannot connect to the Aura Pulse after multiple attempts, try turning on airplane mode on your mobile phone. Then wait for 5 seconds,

turn back off airplane mode, and close the AuraTrack app. After that, try opening the app again and connecting to the Aura Pulse.
4. If that does not work, try restarting your phone.
5. If all else fails, try restarting the Aura Pulse.

• **Does Bluetooth need to be always enabled when I wear the Aura Pulse?**
In order for the AuraTrack app to function optimally, Bluetooth should always be enabled.

• **What can I do if my Aura Pulse doesn't update?**
There are a few things you can try:
1. First, make sure that the Aura Pulse's battery is above 50% before trying to update it.
2. If the update still doesn't work, just try updating again at a later time.
3. Another option is to try using the tracker while the AuraTrack app is closed on your mobile phone.

• **My Aura Pulse won't charge or turn on after being charged. What can I do?**
First, check to see if the charging cradle is properly connected to the Aura Pulse. If it is, try checking to see if the Aura Pulse is charging after 5-10 minutes by looking at the display. If the Aura Pulse's battery is above 3%, it should turn on normally..

 This symbol means that according to local laws and regulations your product and/or its battery shall be disposed of separately from household waste. When this product reaches its end of life, user has the choice to give his product to a competent recycling organization. Proper recycling of your product will protect human health and the environment.

Warranty



Aura Medical LLC provides a 1 year warranty for your Aura Pulse.

To register your product for warranty please visit
www.aura-well.com/warranty.

Filing for a warranty claim will require:

A copy of the purchase receipt from an authorized retailer

For general support or inquiries please contact us at:
support@aura-medical.com or 888.433.6916

Manufactured for Aura Medical LLC
Made in China



Manufacturer: Guangdong Transtek Medical Electronics Co., Ltd. Address: Zone A,
No.105, Dongli Road, Torch Development District, 528437 Zhongshan, Guangdong, China